AchilloStabil® Plus Sport Achillessehnenbandage

Achilles' Tendon Support | Chevillère de soutien du talon d'Achille
Ortesis para el tendón de Aquiles | Fascia per il tendine di Achille

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The BORT AchilloStabil® Plus Sport Achilles' Tendon Support facilitates damaged tissue structures. The brace compression supports the holding function of the ligaments, thus relieving the joint. At the same time, the brace helps to avoid incorrect movement.

The viscoelastic pad in the Achilles tendon area encourages lymphatic drainage and blood circulation while massaging the area. Swelling and bruising are reduced.

COOLMAX® regulates the body temperature to provide a dry and pleasant feeling on the skin.

Instructions for use:

- Firmly grasp the top of the brace and pull it over the foot.
- Ensure that the Achilles tendon is completely surrounded by the moulded insert.
- The brace fit should be corrected if necessary

Indications:

Chronic or post-traumatic irritation or tenderness in the Achilles tendon area, for example in the case of Haglund syndrome heel irritation, achillodynia, after surgery, for example in the case of Achilles tendon rupture, bursitis subachillea.

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

38% Elastodies, 34 % polyester, 19%polyamide, 9% spandex

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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Hand-wash the support in lukewarm water using a mild detergent. Do not use fabric softeners. Shape by hand and allow dry in the open air. Avoid heat.

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The **BORT AchilloStabil® Plus Sport** combines the relief provided by an Achilles heel support with the moisture regulating properties of COOLMAX® high-tech fibre. Innovative materials are used for manufacture using the latest production technology to give the BORT AchilloStabil® Plus Sport the ideal properties for use in sports and therapy.

BORT-Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.