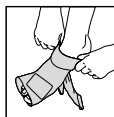
E

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
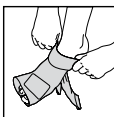
**GB**

The BORT Helix S SPIRALDYNAMIK® USG-Brace supports the natural torsional movement of the foot. The straps attached to the support act by exerting corrective traction on the heel bone towards the middle of the body and support the straightening of the foot arch. Due to the special guiding strap and the compression exerted by the brace, the item supports the holding function of the ligaments and relieves the joints while straightening the arch structures in their correct physiological position. At the same time, the brace helps to avoid incorrect movement. During movement, the knitted compression fabric and the silicone pads exert a massaging effect. The tendency for swelling is reduced and the healing process is accelerated.

#### Instruction for use:

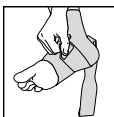1. Loosen both straps and pull the support over the foot up to the ankle area. Then check that the fit is correct. In order to further simplify application we recommended – if possible – leaning the foot on the opposite thigh

2. Gently stretch the red strap around the outside of the heel and the sole of the foot towards the base joint of the big toe. Secure the top of the Velcro fastening to the red velour surface at the back of the foot.

3. Gently stretch the grey strap around the ankle and across the top of the foot. If necessary, the heel bone can be pulled with the other hand towards the middle of the body in order to support the strap at this stage.

4. Guide the grey strap under the sole of the foot and across the inside edge of the foot.

5. Fasten the grey strap by securing the top of the Velcro fastening to the green strap at the back of the foot, while stretching gently. Check and, if necessary, correct the fit and the individual strap tension of the item.

#### Composition:

65% Polyamide 28% elastodies, 7% viscose

#### Indications:

Chronic, post-traumatic or post-operative irritations of soft tissues in the area of lower ankle joint and the sole of the foot, for example in the case of arthrosis, ligament weakness, foot problems due to static anomalies and defects in the shape of the foot due to dysfunction of the lower ankle joint.

#### Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

#### Important instructions:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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