E

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ⒼⒷ

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated.

#### **Instructions for use:**

Before putting on the BORT KubiFX Light Elbow Brace, it will be necessary to shape the two aluminum splints to match the desired joint position. Then check the positioning of the elbow pad and insert the fastening straps through the loops. Now put the brace on your arm, making sure that the tip of the elbow (olecranon) is properly positioned in the curvature of the aluminum splints. Fasten the straps with the desired degree of tightness. The two middle straps cross one another distally to the bend of the elbow in the area of the lower arm and thereby do not compress the nearby nerves and blood vessels. Now check to make sure that the elbow joint is fixed in place without cutting off the flow of blood.

#### **Indications:**

Pre-operative, post-operative, post-traumatic, for example in the case of proximal sulcus ulnaris syndrome, insertional tendinopathy, post-bursectomy conditions .

#### **Contraindications:**

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

#### **Composition:**

40% PUR foam, 30% cotton, 30% polyamide

#### **Important instructions:**

No reutilisation – this medical device is meant for the care of one patient only.  
In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary. If complaints continue, please contact the doctor.

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