(E)

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GB

Back braces support injured tissue structures (bones, ligaments, capsules and cartilage). The holding function of the ligaments is supported by compression, while the vertebrae are relieved.

The pad operates a massaging effect, thanks to which the tendency for swelling is reduced or existing swelling is rapidly relieved.

Instruction for use:

Place the support around the waist and, with the help of the hand straps, fix both sides uniformly one above the other while slightly pulling together. Make sure that the support is placed centrally and ensure that the bottom edge of the support ends in the groin area (Sit-ups).

This is the only way to fully benefit from the effect of the BORT Activemed Back Support. The pad is attached to the support with Velcro and can be freely positioned. Make sure that the broad side of the pad is positioned towards the head.

Composition:

63% Polyamide, 22% elastodies, 15% spandex

Indications:

Treatment of pain in the lumbar spine, low back pain, recurrent low back pain, osteochondrosis, spondylarthrosis, spondylolysis without spondylolisthesis, mild lumbar deformity (degenerative spondylolisthesis).

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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