E

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GB

Immobilisation is a basic principle in the treatment of injuries and overloading, as the healing processes are significantly accelerated. Wrist braces support injured tissue structures (joints, ligaments, capsules, cartilage). The brace supports the holding function of the ligaments, thus relieving the joint. The inserted aluminium splint is shaped in such a way as to fix the wrist and thumb in a relaxed functional position. Furthermore, the brace provides the user with enhanced protection against incorrect movement.

Instruction for use:

Before application, curve the aluminium splint into the desired anatomical shape. If necessary, the thumb portion can be positioned individually on the wrist brace using the Velcro fastenings. Insert the hand into the brace. Next place the medium Velcro strap around the wrist and fasten, then close the Velcro strap at forearm and on the back of the hand. Lastly, close the Velcro fastenings in the thumb area; check the fit of the brace and adjust if necessary.

Composition:

83% Polyester, 17% polyester microfibre

Indications:

Pre-operative, post-traumatic and post-operative conditions, i.e. after thumb ligament reconstruction, immobilisation in the case of rheumatic-inflammatory processes, i.e. rhizarthrosis, in the case of degenerative diseases such as de Quervain's Disease, carpal tunnel syndrome.

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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