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GB

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. Due to the thumb opening the support does not slip. The highly elastic form-knit allows easy support application and yet provides a good supporting effect.

Instruction for use:

Grasp the edge of the support and pull it over the hand. The opening for the thumb prevents the support from slipping

Composition:

68% Polyamide, 25% elastodies, 7% viscose

Indications:

Chronic, post-traumatic or post-operative irritations in the area of saddle joint of the thumb, i.e. in the case of ulnar and radial ligament instability, early stages of rhizarthrosis

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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