EpiBasic Sport Epicondylitis-Bandage

Epicondylitis Support | Bandage Epicondylitis
Vendaje Epicondilitis | Gometiera per epicondillite

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Elbow Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. The compressing knit and the silicone pad provide a massage effect, accelerating the healing process. Targeted pressure alleviates pain and relieves traction on the tendon origins.

COOLMAX® regulates the body temperature to provide a dry and pleasant feeling on the skin.

Instructions for use:

Apply the support such that the knitted comfort area is positioned in the crook of the arm. The support is properly fitted if you can feel the outer and inner joint extension through the cut-outs of the silicone pad.

Indications: Chronic post-traumatic or post-operative irritations of knee joint soft tissues, recurrent articular effusion, i.e. in the case of gonarthrosis, arthritis, ligament instability, meniscopathy.

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

38% Elastodies, 34% polyester, 19% polyamide, 9% spandex

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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
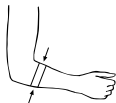
xx-small	_____	20 – 22 cm
x-small	_____	22 – 24 cm
small	_____	24 – 26 cm
medium	_____	26 – 28 cm
large	_____	28 – 30 cm
x-large	_____	30 – 32 cm
xx-large	_____	32 – 34 cm

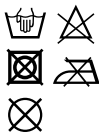Pflegehinweis / Washing instruction / Instructions de lavage / Instrucciones para el cuidado / Manutenzione

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Hand-wash the support in lukewarm water using a mild detergent. Do not use fabric softeners. Shape by hand and allow dry in the open air. Avoid heat.

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The **BORT EpiBasic Sport Elbow Support** combines the wearing comfort of a conventional epicondylitis support with the benefits of the COOLMAX® functional fibre. Innovative materials combined with cutting edge production technology endow the BORT EpiBasic Sport with optimal properties to specifically influence the affected tendon insertions during therapy. The healing process is continuously supported and promoted.

BORT-Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.